

REF - MPC

LOCO-009

Transfer Sheet 190cm x 100cm

LOCO-205

Deluxe Transfer Sheet 190 x 100cm



⚠ All patient moving and handling tasks should be carried out in accordance with approved patient moving and handling techniques and after a full assessment of patient, equipment, task and environment. Always follow the approved policy of your place of work when adopting moving and handling practices.

Description

The Locomotor Transfer Sheets are full body length washable transfer sheets incorporating handholds to enable sliding transfers. Typically used in combination with a bridging board, such as the Lateral Transfer Slide LOCO-119, for lateral transfers from bed to trolley etc.

Locomotor Transfer Sheets may also be used for floor transfers typically sliding a patient to a different location for physiotherapy mat work or evacuation.

Both the Transfer Sheet and the Deluxe Transfer Sheet have a 127kg weight limit for emergency vertical lifts where a hoist is either unavailable or inappropriate*.

Instructions for use

Lateral transfers incorporating a lateral transfer slide / board

Using an approved rolling technique position the transfer sheet centrally underneath the patient. Whilst the patient is being rolled onto the transfer sheet insert a lateral transfer slide underneath the transfer sheet in the correct position. Ideally with a third of the transfer slide's width under the patient and third of its width available for the destination surface.

Position the starting surface (e.g. bed) and destination surface (e.g. trolley) so that they are tightly aligned with minimal gap and with brakes applied to both surfaces. Have the starting surface slightly higher than the destination surface and then slide the patient to the destination surface using approved techniques and the required number of personnel.


Floor transfers


Having assessed it safe to move the patient, roll them onto the transfer slide using an approved rolling technique. Place a pillow or similar underneath the patient's head (on top of the transfer sheet).

Using the required number of personnel, utilise the most appropriate handholds and slide the patient to the required destination.

Emergency lifts*

Roll the patient on to the transfer sheet and follow your manual handling protocol for emergency lifts ensuring you have the correct number of personnel and appropriate equipment to facilitate a vertical lift onto a horizontal surface (ideally a very low height adjustable trolley).


 * This must be in accordance with your moving and handling policy, training and risk assessment. If emergency vertical lifts are not accommodated in your establishment, then do not perform.


 **For further information and training on moving and handling techniques please contact your Moving and Handling Advisor or similar.**


Care and use

 Before each use inspect item. **Do not use** if the product is soiled or damaged

 Washing temperature up to 71°C


 Do not bleach


 Do not dry clean

 May be tumble dried on cool setting

 Maybe drip dried

Storage


 Store away from direct sunlight and sources of heat


 Store in dry conditions

Specifications

Materials: Nylon woven body with Polyester webbing handholds

Safe working load (SWL): 127 kg

 Does not contain latex


Disposal:  May be recycled where facilities exist

Warranty:



Warranted to specification for 12 months from date of purchase.

Applied regulations and standards

MD Class 1 Medical Device UK MDR 2002 EU MDR 2017/745 BS EN 12182: 2012 Tech Aids for disabled persons	
--	---

For further assistance please contact your supplier or customer services at:



Manufacturer





Select Healthcare (UK) Ltd.
 Unit 2 Albert Road
 Rushden
 Northants
 NN10 0BU

Tel: +44 (0) 1933 411729
 Fax +44 (0) 1933 410213
 Email: info@selecthealthcare.co.uk
 Website:
www.selecthealthcare.co.uk

European Representative

EC REP Select Healthcare (EU), 72 Liffock Park, Coleraine, Northern Ireland, BT51 4DG

Glossary of symbols and terms used in product and packaging labels that are not expressed above.

Symbol	Meaning	Associated Terms
	Consult instructions	
DI	Device Identifier – (GS1 UK GTIN 13 used)	AI (01) – Application identifier 01
SN	Serial Number	AI (21) – Application identifier 21
LOT	Lot / Batch Number	AI (10) – Application Identifier 10
	Date of manufacture	AI (11) – Application Identifier 11
UDI	Unique Device Identifier - collective term for DI, SN, LOT & date of manufacture (all contained within machine readable GS1 DataMatrix)	GS1 DataMatrix = 2D Matrix symbols that can carry all GS1 keys and attributes

